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IX. Technical data

Length of the base	35 cm
Regulated height with the Face Rest	58-72 cm
Depth	50 cm
Weight without accessories	4 +/- 1 kg
Maximum load allowed	150 kg

instruction manual for the rehabilitation chair

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I. Application

Mobile Matt is used for the treatment or soothing the course of a disease and the effects of injuries and handicap by allowing to perform effective and safe rehabilitation and therapeutic massages.

II. Design

1. Frame
2. Upholstered Chest Pillow
3. Adjustable Face Rest
4. Front Shelf which supports the equipment
5. Adjusting mechanism for setting the angle of the Face Rest and Front Shelf
6. Carry Case

III. Operation

1. Pull out Mobile Matt from the carry case.
2. Find support point for the Front Shelf (4) providing proper stability of the equipment and comfort for a patient (the height of the point should be proper for the height of the patient).
3. According to support point specification set the angle of the Front Shelf (4).
4. Use knobs to set desire height and fix to frame (1) regulated Face Rest (3).
5. Use the Adjusting mechanism for setting the angle (5) of the regulated Face Rest (3).

Placing the patient:

The patient in the sitting position leans forward, puts his hand on support element of the equipment and places his face in the pillow of regulated Face Rest.

IV. Disassembly of the Mobile Matt


1. Disassembly the Face Rest (3).
2. Fold the equipment and put it to carry case (6).
3. Put the Face Rest (3) in a pocket of carry case (6).

V. Maintenance and cleaning

Chromium-plated and lacquered elements. For cleaning such elements products that do not contain abrasives are recommended. Exemplary disinfectants: Incidur Spray, Incidin Foam Spray, tissues Sani Cloth Active.

VI. Disinfection

Disinfect when necessary and sporadically.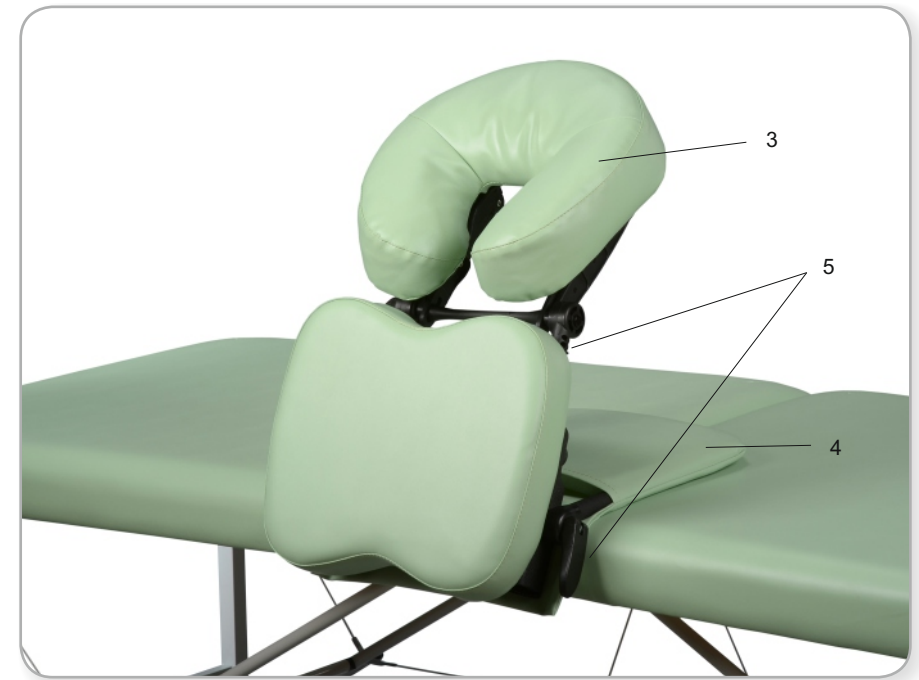

VII. Recommendations

1. The equipment should be used in accordance with the instruction manuals.
2. It is necessary to get acquainted with the terms of warranty and observe them.

VIII. Exploitation remarks

1. Mobile Matt is intended for use inside the dry and closed rooms and for the treatment of one person only at a time.
2. Mobile Matt should be used and stored far away from the direct sources of fire and heat.
3. Correct use of Mobile Matt will guarantee its long and failure-free exploitation.
4. Damages that arose as a result of noncompliance of the above-mentioned instructions do not constitute a basis for a complaint.